

GENTLE TOUCH HQ GREEN

NITRILE 6.0 GR



NITRILE	GREEN	POWDER FREE	S-XL	AMBIDEXTROUS	NON-STERILE
---------	-------	-------------	------	--------------	-------------

Comfortable glove. It offers protection against specific chemical hazards and sporadic contact solvents. Its uses include sanitary, examination and examination, laboratory, hygiene and cleaning, hairdressing, tattoo artists, automotive, DIY, food, etc.

Features:

It does not contain latex protein, suitable for latex allergy sufferers. Type I.	Textured at the tips, making it easier to grip.
Green, dust-free, single-use, ambidextrous, non-sterile.	High strength and durability.
Cuff with flange, spill protection.	For superior protection. Adaptability.

Presentation:

- Dispenser box of 100 units, with die-cut opening for hygienic access to the glove.
- Packaging contains: 10 boxes of 100 units
- Dimension: 250 x 130 x 75 mm



Labelling: Expiry date, date of manufacture, batch number, Reference, barcode. Product name in several languages: Spanish, English, French, German, Portuguese, Italian.

CE marking, Protection pictograms, Safety standards and legislation. Size and units, size guide.

Glove Dimensions	S	M	L	XL
PALM WIDTH	85 ± 5 mm	95 ± 5 mm	110 ± 5 mm	110 ± 5 mm
LENGTH	Min 240 mm	Min 240 mm	Min 250 mm	Min 250 mm
Palm thickness mm (± 0.02)	0.12	0.12	0.12	0.12
Finger thickness mm (± 0.02)	0.17	0.17	0.17	0.17
Cuff thickness mm (± 0.02)	0.09	0.09	0.09	0.09
Weight gr. (± 0.3)	5.8	6.0	6.3	6.5

RUBBEREX SPAIN S.L

+34 96 166 7124

info@rubberex.es

www.rubberex.es

Masía del Baló Industrial Estate. C/Mas Vallesa de Mandor N8 Nave12. CP. 46190. Ribarroja del Turia. Valencia. Spain

GENTLE TOUCH HQ GREEN

NITRILE 6.0 GR



Regulations:

- Category III. CE mark according to Regulation (EU) 2016/425.
- Food regulation 10/2011 on plastic materials intended to come into contact with food. **Regulation (EC) No 2023/2006.** Good Manufacturing Practices for Materials and Articles Intended for Contact with Food and Regulation **(EU) 2025/351.** **Regulation (EC) No 1935/2004:** Material and articles in contact with food. . **Regulation (EU) 2024/3190** of 19 December 2024, which repeals R EU 2018/213. BPA. Bisphenol A*: CAS 80-05-7.
- EN ISO 374-1:2016+A1:2018. Point 7 EN 374-2. Point 5 of the EN374-4 standard. Method EN 16523-1:2015.
- EN ISO 374-5:2016. Risks of microorganisms (fungi, viruses and bacteria).
- EN420:2003+A1:2009. EN ISO 21420:2020.
- EN455-1-2-3-4. Breaking Strength 12.76 N.
- EN ISO 18889:2019- Protection against pesticides and workers on G1 farms.
- This product is not classified as hazardous according to Regulation EC 1272/2009 of the European Parliament and of the Council.
- This product contains less than 0.1% substances of very high concern (SVHC) and does not contain substances included in Annex VII of Regulation 1907/2006 of the European Parliament and of the Council.

CHEMICAL RESISTANCE

PERMEATION. EN ISO 374-1:2016/A1:2018

Gloves are classified in terms of passage time, depending on each individual chemical for which the glove resists permeation.

The gloves are tested against the following chemicals:

CHEMICALS	LEVEL	LYRICS
N-HEPTANE	6	J
SODIUM HYDROXIDE 40%	6	K
HYDROGEN PEROXIDE 30%	4	P
FORMALDEHYDE 37%	6	T

Performance levels:

Passage time (min)	> 10	> 30	> 60	> 120	> 240	> 480
Performance level	1	2	3	4	5	6

RUBBEREX SPAIN S.L

+34 96 166 7124

info@rubberex.es

www.rubberex.es

Masía del Baló Industrial Estate. C/Mas Vallesa de Mandor N8 Nave12. CP. 46190. Ribarroja del Turia. Valencia. Spain

GENTLE TOUCH HQ GREEN NITRILE 6.0 GR



DEGRADATION

The % degradation (DR) is determined for each chemical made in the permeation.

(DR) J: n-Heptane = 34.4%

(DR) K: Sodium hydroxide 40% = -5.0%

(DR) P: Hydrogen Peroxide 30%: 13.2%

(DR) T: Formaldehyde 37% = 27.05%

The PPE type GENTLE TOUCH HQ NITRILE GLOVE should be used to protect the wearer's hand against chemical hazards (products and levels mentioned above) and against microbiological hazards (bacteria, fungi and viruses).

This PPE should never be used against risks other than those described above. The use of this PPE should be evaluated correctly depending on the workplace.

EN ISO 374-1:2016+A1:2018 / Type B EN ISO 374-5:2016



J K P T



VIRUS



CAT III



ISO 18889 G1

REF	Color	Size	T. numerical	EAN
0404	Green	S	7	8435234504048
0405	Green	M	8	8435234504055
0406	Green	L	9	8435234504062
0407	Green	XL	10	8435234504079



250x75x130mm



Logistics:

Dispenser: 100 gloves

Box: 10 dispensers per box

Pallet: 56 boxes

RUBBEREX SPAIN S.L

Rev 003 2026

+34 96 166 7124

info@rubberex.es

www.rubberex.es

Masía del Baló Industrial Estate. C/Mas Vallesa de Mandor N8 Nave12. CP. 46190. Ribarroja del Turia. Valencia. Spain